



## Redefine Your Preclean Performance with INTERCEPT™

The **INTERCEPT™ Detergent and Foam Pre-Treatment process** enables hospitals to meet changing conditions and difficult time constraints by increasing the current recommendation of time between precleaning and manual cleaning from 1 hour to up to 72 hours.

INTERCEPT™ Detergent & Foam Provides Superior Preclean Performance		
	Offers peace of mind in meeting department protocols	Extend the time between precleaning and manual cleaning of flexible endoscopes without compromising the ability to clean properly
	Increase efficiency by eliminating the risk of delayed scope reprocessing	INTERCEPT™ Foam maintains a moist environment eliminating the need for harmful extended soaks with unclear time requirements
	Simplifies transporting of endoscopes between facilities or departments	Thick foam consistency eliminates liquid sloshing or spills during transportation and prevents soil from drying in or on flexible endoscopes while waiting to be manually cleaned
	Reduce costs associated with extended staffing to clean endoscopes	Endoscopes used in evening and weekend procedures can be pre-treated with INTERCEPT™ Detergent and Foam and left until regular business hours
	Improve procedure outcomes and patient safety	Foam encapsulates an endoscope creating a moist environment to prevent organic debris from drying and hardening

# The INTERCEPT™ Detergent and Foam Pre-Treatment Process

The INTERCEPT™ Detergent and Foam pre-treatment extends the time between precleaning and manual cleaning without compromising the ability to clean properly. **Here's how:**

## 1. Preclean your endoscope using INTERCEPT™ Detergent

- INTERCEPT™ Detergent is a non-enzymatic, surfactant-based formulation designed to assist in the breakdown of biofilm bonds



## 2. Completely Encapsulate Endoscope in INTERCEPT™ Foam

- Place the endoscope into a rigid sealable container, such as the CLEANASCOPE™ Tray
- The power of INTERCEPT™ in a thick stable foam maintains a moist environment preventing organic debris from drying and hardening



## 3. Seal the CLEANASCOPE™ Tray with CLEANASCOPE™ Liner

- The CLEANASCOPE™ Liner creates a seal and clearly indicates contaminated contents
- The rigid tray protects the endoscope during transportation



Part Number	Description
ML02-0133	INTERCEPT™ Detergent (4 one-gallon bottles/case)
ML02-0107	INTERCEPT™ Wipes (4 containers/case, 100 wipes/container)
ML02-0139	INTERCEPT™ Foam (12 ounce bottles/case)
100900	INTERCEPT™ Bedside Kit (box of 50 kits containing detergent packet, sponge and disposable basin)
Consult Your Sales Representative	CLEANASCOPE™ Transport System, CLEANASCOPE™ Liners & Accessories Customized procedure kits



INTERCEPT™ is a trademark of Medivators Inc.  
CLEANASCOPE™ is a trademark of Cantel (UK) Limited.



**Cantel (Canada) Inc.**  
90 Gough Road, Unit 1  
Markham, ON L3R 5V5

Phone: +1.844.348.5636  
Fax: +1.844.348.5637  
Email: info@cantelmedical.ca

[www.cantelcanada.com](http://www.cantelcanada.com)

© 2019 Medivators Inc. P/N 50098-1601-ENCAD/A