

SCOPE BUDDY™ PLUS

Endoscope Flushing Aid



A REVOLUTIONARY SOLUTION


TO AN EVOLUTIONARY PROCESS

SCOPE BUDDY™ PLUS Endoscope Flushing Aid helps create a consistent, reliable, and safe means to flush endoscope channels. It provides disposable tubing, increased data monitoring, and automated flushing modes to properly flush endoscopes.

Manual cleaning* is the most important step in reprocessing endoscopes, ultimately helping to reduce the risk of Healthcare-associated infections.¹

Over 220,000 Canadians contract HAIs/year
Over 8,000 Canadians die from HAIs/year²

* SCOPE BUDDY™ PLUS Endoscope Flushing Aid is only intended to be used as a Flushing Aid during the Manual Cleaning step of endoscope reprocessing. It is not intended to manually clean endoscopes.

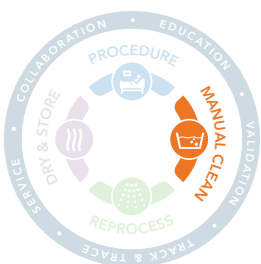

THE COMPLETE CIRCLE OF PROTECTION

As the global vanguard in infection prevention, **only Cantel Medical delivers the Complete Circle of Protection**, a full-value, proactive partnership dedicated to helping you remove risk, streamline operational efficiencies and optimize your success.

MANUAL CLEAN Manual cleaning is essential in removing bioburden and preparing the endoscope for high-level disinfection. For optimal results, Cantel's manual cleaning products and detergents are designed to be a critical first step to efficiently and effectively eliminating bacteria on endoscopes prior to reprocessing.



BEST PRACTICE IS OUR PROMISE

*Industry Standards of Infection Prevention in Reprocessing Flexible GI Endoscopes confirm manual cleaning of endoscopes is necessary prior to high-level disinfection.**

- Standardizing the process of cleaning endoscopes by utilizing both manual and automated methods will help ensure optimal cleaning while complying with the CSA, AAMI, Regulatory and Society Standards and Guidelines
- Recent investigations conclude that manual cleaning is clearly not being performed consistently or correctly resulting in serious negative patient outcomes^{3,4}
- Automated flushing pumps should be used to replace manual syringing. Automated flushing can reduce the risk of repetitive injury and exposure to liquids due to splashing. Use of automated pumps ensures a standardized and therefore consistent process for flushing of all endoscope lumens⁵

Progressively Automated Thoroughly Validated

INCREASING SAFETY

- **Reduce cross contamination risks** by using 24-hour disposable tubing instead of reusable tubing
- **Continuously monitor** the endoscope manual flushing process with the Track and Trace features
- **Validated 1:35 default flushing time** to meet worst case flushing times for all major endoscope brands

IMPROVING EFFICIENCY

- **Automatic and Manual Modes** provide easy to use steps for a dependable flushing process
- **Compatible** with all major endoscope brands
- **Improves staff efficiency** by simultaneously flushing multiple endoscope channels

IMPACTING LIVES

- **Best practice flushing standards** help reduce the risk of hospital acquired infection breakouts for patients
- **Consistent, hands-free flushing** replaces repetitive and labor-intensive syringing of channels



**Continuing
our 40 year
legacy in
infection
prevention**



THE MOST EXTENSIVE MANUAL CLEANING PORTFOLIO

- INTERCEPT™ Bedside Kit
- INTERCEPT™ Wipes
- VERISCAN™ LT Automated Endoscope Leak Detection System
- SCOPE BUDDY™ PLUS Endoscope Flushing Aid
- INTERCEPT™ Detergent



SCOPE BUDDY™ PLUS Endoscope Flushing Aid Ordering Information

SCOPE BUDDY™ PLUS ENDOSCOPE FLUSHING AID

MODEL NUMBER	DESCRIPTION
SBP-1000	SCOPE BUDDY™ PLUS Endoscope Flushing Aid

DETERGENT

PART NUMBER	DESCRIPTION
ML02-0133	INTERCEPT™ Detergent

ACCESSORIES

SB-110-HU1015	24-Hour Multi-Use Pump Tube
SB-110-HU1007	24-Hour Multi-Use Hookup for Olympus® Endoscopes
SB-110-HU1008	24-Hour Multi-Use Hookup for PENTAX® Endoscopes
SB-110-HU1009	24-Hour Multi-Use Hookup for FUJIFILM™ Endoscopes
SB-110-HU1010	24-Hour Multi-Use DSD Hookup Adapter
SB-110-HU1016	24-Hour Multi-Use Universal Hookup
SB-110-HU1012	Auxiliary Detergent Out Tube (24-Hour Multi-Use)
SB-110-HU1011	Detergent In Tube—Change with new detergent
SB-110-HU1017	Olympus® Air/Water Blank Cap



SCOPE BUDDY™ PLUS Endoscope Flushing Aid

Unit Dimensions

10.3 H x 7.5 W x 11.5 D (inches)

26.2 H x 19 W x 29.2 D (cm)

Unit Weight

10 lbs (4.5 kg)

Warranty

15 Months from the date of shipment, or one year from date of installation, whichever comes first.

Electrical Requirements

100 – 240 V, 50/60 Hz

- 1 Rutala, W.A., Weber, D. J., and the Healthcare Infection Control Practices Advisory Committee (2008). *Guideline for Disinfection and Sterilization in Healthcare Facilities* (Last update: May 2019). Retrieved from CDC: <https://www.cdc.gov/infectioncontrol/pdf/guidelines/disinfection-guidelines-H.pdf>
- 2 Canadian Patient Safety Institute. *Healthcare Associated Infections (HAI)*. Retrieved from CPSI: [https://www.patientsafetyinstitute.ca/en/Topic/Pages/Healthcare-Associated-Infections-\(HAI\).aspx](https://www.patientsafetyinstitute.ca/en/Topic/Pages/Healthcare-Associated-Infections-(HAI).aspx)
- 3 FDA Safety Communication Design of Endoscopic Retrograde Cholangiopancreatography (ERCP) Duodenoscopes May Impede Effective Cleaning. September 2018. <https://www.fda.gov/medical-devices/reprocessing-reusable-medical-devices/infections-associated-reprocessed-duodenoscopes>
- 4 Zachary L. Smith, Young S. Oh, Kia Saeian, Charles E. Edmiston Jr, Abdul H. Khan, Benson T. Massey, Kulwinder S. Dua., 2015. Transmission of carbapenem-resistant Enterobacteriaceae during ERCP: time to revisit the current reprocessing guidelines. *Gastrointestinal Endoscopy* 81(4) pp.1041-1045
5. National Standard of Canada. (2018). *Canadian medical device reprocessing – 12.4 Reprocessing of endoscopes (CAN/CSA-Z314-18)*. Retrieved from <https://www.scc.ca/en/standardsdb/standards/29301>

*SCOPE BUDDY™ PLUS Flushing Aid is only intended to be used as a Flushing Aid during the Manual Cleaning step of endoscope reprocessing. It is not intended to manually clean endoscopes.

SCOPE BUDDY™, INTERCEPT™ and VERISCAN™ are trademarks of Medivators Inc. All other company and product names are trademarks of their respective owners.

MEDIVATORS IS NOW CANTEL

www.cantelcanada.com

TO PLACE AN ORDER

p: 1.844.348.5636 f: 1.844.348.5637 e: orders@cantelmedical.ca

