

The INTERCEPT™ Detergent and Foam Pre-Treatment Process

The INTERCEPT™ Detergent and Foam pre-treatment extends the time between precleaning and manual cleaning without compromising the ability to clean properly. **Here's how:**

1. Preclean your endoscope using INTERCEPT™ Detergent

- INTERCEPT™ Detergent is a non-enzymatic, surfactant-based formulation designed to assist in the breakdown of biofilm bonds



2. Completely Encapsulate Endoscope in INTERCEPT™ Foam

- Place the endoscope into a rigid sealable container, such as the CLEANASCOPE™ Tray
- The power of INTERCEPT™ in a thick stable foam maintains a moist environment preventing organic debris from drying and hardening



3. Seal the CLEANASCOPE™ Tray with CLEANASCOPE™ Liner

- The CLEANASCOPE™ Liner creates a seal and clearly indicates contaminated contents
- The rigid tray protects the endoscope during transportation



| Part Number | Description |
|-----------------------------------|--|
| ML02-0133 | INTERCEPT™ Detergent (4 one-gallon bottles/case) |
| ML02-0107 | INTERCEPT™ Wipes (4 containers/case, 100 wipes/container) |
| ML02-0139 | INTERCEPT™ Foam (12 ounce bottles/case) |
| 100900 | INTERCEPT™ Bedside Kit (box of 50 kits containing detergent packet, sponge and disposable basin) |
| Consult Your Sales Representative | CLEANASCOPE™ Transport System, CLEANASCOPE™ Liners & Accessories Customized procedure kits |



INTERCEPT™ is a trademark of Medivators Inc.
CLEANASCOPE™ is a trademark of Cantel (UK) Limited.



Cantel (Canada) Inc.
90 Gough Road, Unit 1
Markham, ON L3R 5V5

Phone: +1.844.348.5636
Fax: +1.844.348.5637
Email: info@cantelmedical.ca

www.cantelcanada.com

© 2019 Medivators Inc. P/N 50098-1601-ENCAD/A